
Instructions for use

Dräger Alcotest 4000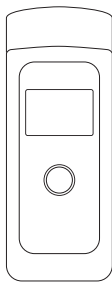

More languages:
draeger.com/ifu


enUS, es

WARNING

To properly use this product, read and comply with these instructions for use.

enUS	Instructions for use	4
es	Instrucciones de uso	22

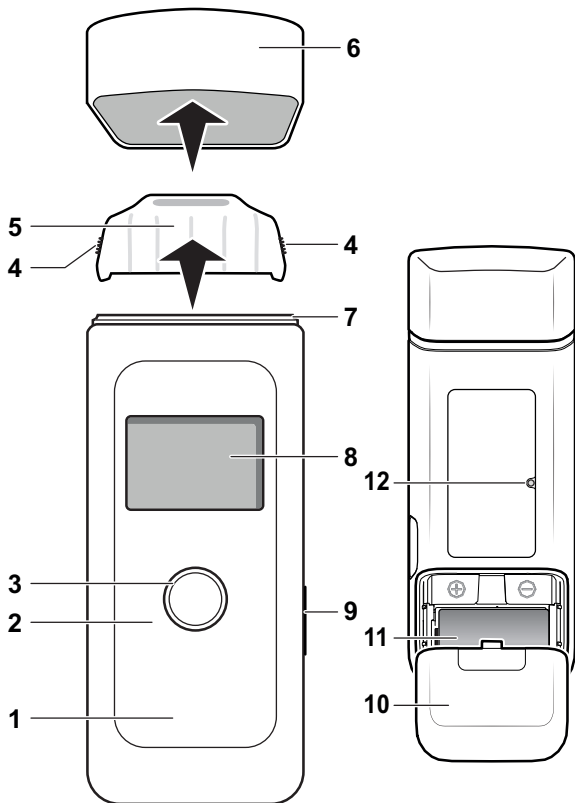Contents

1	Safety information	5
2	Conventions in this document	6
2.1	Meaning of the warning notes.	6
2.2	Typographical conventions.	6
3	Description	7
3.1	Product overview	7
3.2	Intended use.	7
4	Use	7
4.1	Prerequisites for use.	8
4.2	Initial start-up	9
4.3	Carrying out a breath sample	11
4.4	Removing the mouthpiece	12
4.5	Menu control.	12
4.6	Menu items.	13
5	Troubleshooting	16
6	Maintenance	17
6.1	Replacing the battery	17
6.2	Cleaning	17
6.3	Calibration.	18
7	Storage	18
8	Disposal	19
9	Technical data	19

1 Safety information

- Before using this product, carefully read these instructions for use and those of the associated products.
- Strictly follow the instructions for use. The user must fully understand and strictly observe the instructions. Use the product only for the purposes specified in the intended use section of this document.
- Do not dispose of the instructions for use. Ensure that they are stored and used appropriately by the user.
- Only trained and competent users are permitted to use this product.
- Do not use a faulty or incomplete product. Do not modify the product.
- Notify Dräger in the event of any component fault or failure.
- Comply with all local and national rules and regulations associated with this product.
- Only trained and competent personnel are permitted to inspect, repair and service the product. Dräger recommends a Dräger service contract for all maintenance activities and that all repairs are carried out by Dräger.
- Use only genuine Dräger spare parts and accessories. Otherwise, the proper functioning of the product may be impaired.

 These instructions for use are available in further languages and can be downloaded in electronic form on the according product website (www.draeger.com) or a printed version can be ordered free of charge via Dräger or specialist shops.

- Only use battery chargers, power packs, batteries, or rechargeable batteries that Dräger has approved for this product.

2 Conventions in this document

2.1 Meaning of the warning notes

The following warning notes are used in this document to notify users of possible dangers. The meanings of the warning notes are defined as follows:

Alert icon	Signal word	Classification of warning
	WARNING	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
	CAUTION	Indicates a potentially hazardous situation which, if not avoided, could result in injury. It may also be used to alert against unsafe practices.
	NOTICE	Indicates a potentially hazardous situation which, if not avoided, could result in damage to the product or environment.

2.2 Typographical conventions

- Text** Text marked in bold denotes labeling on the device and on-screen messages.
- ▶ This triangle labels possible methods of avoiding the hazards mentioned in warning notices.
- > The greater-than sign denotes the navigational path in a menu.
-  This symbol denotes information that make using the product easier.

3 Description

3.1 Product overview

See Fold-out Page:

1	Breath alcohol testing device
2	LED ring
3	Button
4	Grip zone on mouthpiece
5	Mouthpiece
6	Safety cap
7	Mouthpiece receptacle
8	Display
9	USB connection (Important: Not for charging the device, for service only)
10	Battery compartment cover
11	Battery
12	Loudspeaker (Important: No reset button)

3.2 Intended use

The Dräger Alcotest[®] 4000¹⁾ breath alcohol testing device is used to measure the alcohol concentration in human breath. The measurement should not be used as grounds for medical or therapeutic treatments.

4 Use

WARNING

Small parts of the device may become detached and be swallowed.

- ▶ The device is not intended for children under 3 years.

1) Alcotest[®] is a registered trademark of Dräger.

4.1 Prerequisites for use

The device is factory-adjusted when delivered. If the adjustment date is longer ago than 12 months, the device should be adjusted, e.g., by DrägerService.

Optional: If the adjustment date has been exceeded, a warning message is displayed after the startup screen. Confirm the warning message with a short press on the button or wait some seconds to continue.

- Observe a wait time of approx. 15 minutes after last alcohol consumption.
- The wait time cannot be reduced by rinsing the mouth with water or with non-alcoholic beverages.
- Residual mouth alcohol or aromatic beverages (e.g., fruit juice), alcoholic mouth sprays, medication and drops, as well as belching and vomiting may cause incorrect measuring values.

4.1.1 Before sampling

- The subject must breathe regularly and calmly before sampling.
- The subject must be able to provide the required minimum breath volume for the sampling.

4.1.2 Preparation for use

1. Ensure that the battery is sufficiently charged and correctly inserted.
2. Ensure that a clean mouthpiece is attached.

4.1.3 Switching the device on

1. Hold down the button for about 1 second.
2. Wait until the device displays that it is ready to measure.

4.1.4 Switching the device off

1. Hold down the button for about 3 seconds.

A switch off count down will be shown as well. If the display is off, stop pressing the button. Otherwise the device will be switched on again.

 The device will switch off automatically approx. 2 minutes after the last use.

 Switching off the device does not reduce the waiting period until it is ready to take a measurement!

4.2 Initial start-up

NOTICE

Loss of default settings

If the battery replacement process takes longer than 5 minutes, the date and time must be reset. A measurement is only possible with the time set.

Default wizard

A setup wizard starts when the device is switched on for the first time. This step is not required for pre-configured devices.

The following settings are made using the setup wizard:

- Language
- Date format
- Date
- Time format
- Time
- Measuring unit

All settings can be changed later on via the menu.

Language

A cursor pre-selects one language in the list. Press briefly on the button to change the position of the cursor in order to select your needed language. If the cursor selects the right language please hold the button for approx. 2 seconds to confirm.

Date

A cursor pre-selects the day of the date. Press briefly on the button to change the day until the actual day is shown. Please hold the button for approx. 2 seconds to confirm the day. Please continue with the month and the year in the same way: short press to change and long press to confirm.

Time format

A cursor pre-selects a time format. Press briefly on the button to change the position of the cursor in order to select your needed time format. If the cursor selects the needed time format please hold the button for approx. 2 seconds to confirm.

Time

A cursor pre-selects the hours of the time. Press briefly on the button to change the hour until the actual hour is shown. Please hold the button for approx. 2 seconds to confirm the hour. Please continue with the minutes in the same way: short press to change and long press to confirm.

Measurement unit

A cursor pre-selects a measurement unit in the list. Press briefly on the button to change the position of the cursor in order to select your needed measurement unit. If the cursor selects the needed measurement unit, please hold the button for approx. 2 seconds to confirm.

After the setup wizard is finished, the display shows **Welcome** and continues with **READY** for tests.

The device is now adjusted to your settings and is ready for breath tests.

Of course these and further settings can be adjusted in the device menu (see "Menu items", page 13).

4.3 Carrying out a breath sample

i A new mouthpiece must be inserted for each person who provides a breath test.

For hygiene reasons, hold the mouthpiece only on the grip zones.

i Do not allow any high alcohol concentrations on the sensor, e.g., by rinsing the mouth with a high proof alcohol shortly before the measurement, as this will shorten the service life of the sensor.

4.3.1 Procedure

1. Remove the safety cap from the top of the device.
2. Insert a new mouthpiece (order no. 83 25 250) into the mouthpiece receptacle (see Fold-out Page).
For hygiene reasons, hold the mouthpiece only on the grip zones.
3. If the device is switched off, keep the button pressed for approx. 1 second until the startup screen is displayed.
4. **READY** is displayed after approx. 4 seconds. The device is ready to measure.
5. Take a deep breath and blow evenly and without interruptions into the mouthpiece.
Do not cover the outlet opening on the mouthpiece with your hand. You have to blow a volume of 1.2 liter and at least 1.6 seconds for a valid test. If blowing normally this will take 2-3 seconds. A continuous tone sounds simultaneously and stops when the test parameters are fulfilled and the device has taken the sample. The test result is displayed after a few seconds (depending on the device temperature and the measured concentration).
In case of a negative alcohol test, **0.00** will be displayed. In case of a positive test, the measured alcohol level will be displayed.

4.3.2 Performing another measurement

- To prepare a new measurement, press the button briefly. After the sensor's recovery time (see "Technical data", page 19), **READY** is displayed and the device is ready to use again.

4.4 Removing the mouthpiece

1. Pull the mouthpiece upwards out of the mouthpiece receptacle by holding it on the grip zones on the left and on the right side.
2. Dispose of the mouthpiece in accordance with local regulations.

4.5 Menu control

4.5.1 Displaying the menu

To enter the device menu, the device has to be switched on and it has to show **READY**.

1. Briefly press the button.
A menu symbol and the text **Enter menu** are displayed.
If the button is not pressed for 3 seconds, the device will return to **READY**.
2. Briefly press the button again (within 3 seconds).
⇒ The menu is entered.
If the button is not pressed a second time, the device will return to **READY**.

 The menu is exited automatically after 30 seconds with the button not being pressed.

4.5.2 Navigating in the menu

In the main menu all menu options are listed one below the other.
To change and confirm the selected feature or value:

1. Briefly press the button several times until you reach the intended position.
When the last entry is reached, the next button press will go back up to the top.

- Hold the button for 2 seconds to enter the sub-menu or to confirm the setting.

4.5.3 Exiting the menu

- In the sub-menu, select the **Back** menu item.
In the main menu, select the **Measuring** menu item.

4.6 Menu items

The following table provides an overview of the menu options and how to use them:

Menu option	Description	Interaction instruction
Measuring	Option to leave the menu and to go back to the READY screen to perform a test.	<ul style="list-style-type: none"> Hold the button for 2 seconds to leave the menu.
Results	Overview of the last 10 tests performed on the device (incl. result, date, time, test ID). There is one test displayed per screen.	<ul style="list-style-type: none"> Briefly press the button to navigate through the test result list. Hold the button for 2 seconds to leave the sub-menu.
Infoscreen	Serial no. and part no. of the device	<ul style="list-style-type: none"> Hold the button for 2 seconds to leave the sub-menu.
Next service	Overview, in how many days the device... <ul style="list-style-type: none"> should get serviced should be adjusted should get an accuracy check. 	<ul style="list-style-type: none"> Hold the button for 2 seconds to leave the sub-menu.

Menu option	Description	Interaction instruction
Date & time	Date, time as well as date format (yyyy/mm/dd, yyyy-mm-dd, mm/dd/yyyy, dd.mm.yyyy) and time format (12h, 24h) can be adjusted. The sub-menu can be left via the Back option.	<ul style="list-style-type: none">• Briefly press the button several times until you reach the intended option.• Enter the option by holding the button for 2 seconds.• Change a setting or a set number by briefly pressing the button (e.g. shifting the time).• Confirm a selection by holding the button for 2 seconds.• Select the Back option and hold the button for 2 seconds to leave the sub-menu.
Contrast	The display contrast can be adjusted within 5 levels.	<ul style="list-style-type: none">• Change the setting by briefly pressing the button.• Hold the button for 2 seconds to confirm the setting and leave the sub-menu.

Menu option	Description	Interaction instruction
Speaker	Any acoustical signals of the device can be turned off or to a volume of 50 % or 100 %.	<ul style="list-style-type: none">● Change the setting by briefly pressing the button.● Hold the button for 2 seconds to confirm the setting and leave the sub-menu.
Language	The language of the display can be adjusted.	<ul style="list-style-type: none">● Change the set language by briefly pressing the button.● Hold the button for 2 seconds to confirm the setting and leave the sub-menu.
Measurement unit	The measurement unit for the alcohol test results can be adjusted within the following options: %, ‰, g/L, mg/100 mL, mg%, mg/mL, mg/L, µg/100 mL, µg%, µg/L, g/210 L	<ul style="list-style-type: none">● Change the measurement unit by briefly pressing the button.● Hold the button for 2 seconds to confirm the setting and leave the sub-menu.
Software version	Software version and part number are displayed.	<ul style="list-style-type: none">● Hold the button for 2 seconds to leave the sub-menu.

5 Troubleshooting

A status message is displayed in the event of certain device conditions. The number that appears below the message is used for service functions. If the status message remains after switching the device on and off several times, contact DrägerService or your dealer.

Fault/message	Cause	Remedy
The device cannot be switched on.	Battery drained.	Replace battery.
Volume too low/sample invalid.	Test subject is blowing intermittently or not hard enough.	Blow into the mouthpiece harder and continuously.
Device outside temperature range	Device is too hot or too cold.	Allow the device to cool down or warm up.
↑↑↑ (over range)	Measuring range exceeded. Alcohol residues that are still in the mouth may cause incorrect measuring values.	A wait time of at least 15 minutes after alcohol was last orally consumed must have elapsed (see "Prerequisites for use", page 8). To repeat test, press the button.
Volume too small	Breath sample volume too small.	Repeat breath sample.
Volume too great	Measuring range exceeded.	Repeat breath sample.
Sample interruption	Irregular exhalation, abrupt end, or inhalation	Repeat breath sample.
Fault number	Device fault	Switch the device off and on again. If the fault remains, contact DrägerService or dealer.

Fault/message	Cause	Remedy
... DAYS Until next service!	In ... days next service is needed.	Contact DrägerService or dealer for device service.
... DAYS Service expired!	Since ... days service is needed.	Contact DrägerService or dealer for device service.

6 Maintenance

6.1 Replacing the battery

WARNING

Damage to the device

Do not try to charge any kinds of batteries via the USB port. The device comes without a recharging function.

► Recharge any rechargeable batteries outside the device.

 If the battery replacement process takes longer than 5 minutes, the time must be reset. A measurement is only possible with the time set.

1. Switch off the device.
2. Slide the battery cover off.
3. Replace the battery (battery type: 3 V CR123A; order no.: 83 21 390).
Pay attention to the correct polarity.
4. Fit the battery cover.
5. Check that the time is correctly set.

6.2 Cleaning

The device should be cleaned with an ethanol-based cleaning agent. Cleaning the device:

1. Wipe the device down with a disposable cloth with the cleaning agent.
Do not apply the cleaning agent directly to the device.

Ensure that no liquid enters the gas inlet.

2. After cleaning the device, store it unpacked at room temperature (20-25 °C) for at least 24 hours.
3. Afterward, carry out an alcohol-free breath alcohol test to ensure that no ethanol residue remains in the device and the result **0.00** is displayed.

6.3 Calibration

Dräger recommends to check device accuracy every 12 months. Device calibration and maintenance should only be carried out by authorized persons, e.g. DrägerService.

The last 30 days before next service is needed, a day count down will be shown as information in the display after switching on the device. Please confirm this information with the button. It shall remind you that the device needs to be serviced in the next days. After the service date, the device counts up how many days it is over the requested service. This information can be confirmed by pressing the button as well.

Calibration certificates:

<https://www.draeger.com/productioncertificates>

7 Storage

WARNING

Damage to the device

Leaking battery acid can damage the device.

- Remove the battery if the device will remain unused for an extended period of time.
-

8 Disposal

This product must not be disposed of as household waste. This is indicated by the adjacent icon.

You can return this product to Dräger free of charge. For information please contact the national marketing organisations and Dräger.

Batteries must not be disposed of as household waste. They are therefore marked with the adjacent symbol. Dispose of batteries and rechargeable batteries at battery collection centers as specified by the applicable regulations.

9 Technical data

Measuring principle

Electrochemical sensor

Measuring range

Breath alcohol concentration 0.000 to 0.500 g/210L

Blood alcohol concentration 0.000 to 0.500 % BAC

Small values will be shown as 0.000 to 0.0063 g/210L
0.000

Ethanol mass per blood volume or mass at 20 °C/68 °F and 1013 hPa. For the above data, a conversion factor of 2100:1 is used to calculate the blood alcohol concentration using the breath alcohol concentration data.

Units of measurement

mg/L, µg/100 mL, µg%, µg/L, g/210 L, g/l ‰, %, mg/100 mL, mg%, mg/mL

(country-specific configuration)

Ambient conditions

During operation -5 °C to +50 °C (+23 to +122 °C) 10 to 100 % r. h. (non-condensing) 600 to 1300 hPa

During storage	-20 °C to +60 °C (-4 to +140 °F) 15 to 75 % r. h. (10 to 100 % r. h. <48 h)
----------------	--

Taking samples

Respiratory volume	>1.2 L (default)
Breath flow	>10 L/min (default)
Duration of breath	>1.6 s (default)

Measurement accuracy

Standard deviation of the repeatability with ethyl alcohol standard

<0.50 mg/L	0.008 mg/L or
≥0.50 mg/L	1.7 % of the measured value

<0.100 % BAC	0.0017 % BAC or
≥0.100 % BAC	1.7 % of the measured value whichever value is higher

Wait time for measurement readiness following a previous measurement with:

0.00 mg/L: 2 s	0.00 ‰: 2 s
0.25 mg/L: 20 s	0.050 ‰: 20 s
0.50 mg/L: 40 s	0.100 ‰: 40 s
1.00 mg/L: 80 s	0.200 ‰: 80 s
>1.50 mg/L: 120 s	>0.300 ‰: 120 s

Test for accuracy

Recommendation: every 12 months

Dimensions

50 mm x 133 mm x 29 mm (W x H x D)
1.97 in x 5.27 in x 1.14 in (W x H x D)

Weight

approx. 4.6 oz (130 g)

Power supply

1 x 3 V CR123A

The battery is designed for up to 1500 measurements (depending on use and the ambient conditions).

Protection class

IP 52

Interface

USB connection for service only

CE markings

Electromagnetic compatibility

Configuration possible, see name plate

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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© **Dräger Safety AG & Co. KGaA**
Dräger Safety AG & Co. KGaA
Draeger Inc.